

# How to use EpiPen® and EpiPen® Jr Auto-injectors.

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:

1



- Hold firmly with orange tip pointing downward.
- Remove blue safety cap by pulling straight up. Do not bend or twist.

2



- Swing and push orange tip firmly into mid-outer thigh until you hear a 'click'.
- Hold on thigh for several seconds.



### Built-in needle protection

- When the EpiPen® Auto-injector is removed, the orange needle cover automatically extends to cover the injection needle, ensuring the needle is never exposed.



***After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911.***

For more information visit [EpiPen.ca](http://EpiPen.ca).

EpiPen® and EpiPen® Jr Auto-injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions.

This product may not be right for you. Always read and follow the product label.



© 2013 Pfizer Canada Inc., Kirkland, Quebec H9J 2M5 • Toll free: 1-877-EPIPEN1 (1-877-374-7361)  
EpiPen®, EpiPen® Jr are registered trademarks of Mylan Inc. licensed exclusively to its wholly-owned affiliate,  
Dey Pharma, L.P. of Napa, California USA; sub-licensee, Pfizer Canada Inc., Kirkland, Quebec, H9J 2M5  
0000041397



Trusted for over 25 years.